

Lung VCAR Multi


VolumeShare 7 • Quick Reference Guide

Lung VCAR Multi is an image analysis application that allows the enhanced visual detection of lesions within the lungs. It provides the physician with tools for determination of nodule consistency, volume, diameters and density, as well as the comparison between different exams for the same patient taken over a given period of time to determine the nodule growth rate and estimated doubling time.


- Select **exams** to compare using **Ctrl + left click** in patients list and load in **Lung VCAR Multi**




- Select each exam's **mediastinum series** (*standard or bone*), or current exam's and previous exam's **AutoSave**.

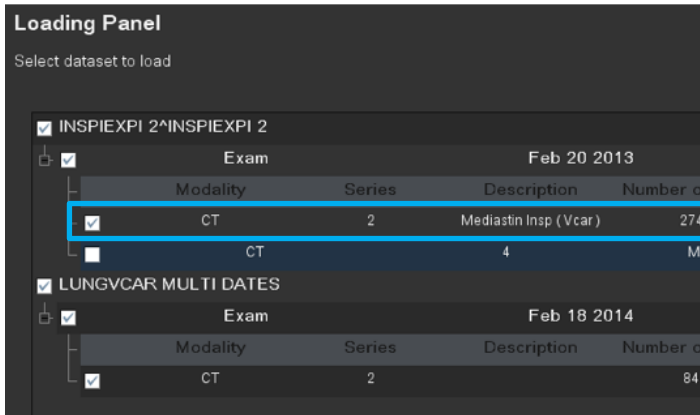

- Exams are loaded this way :

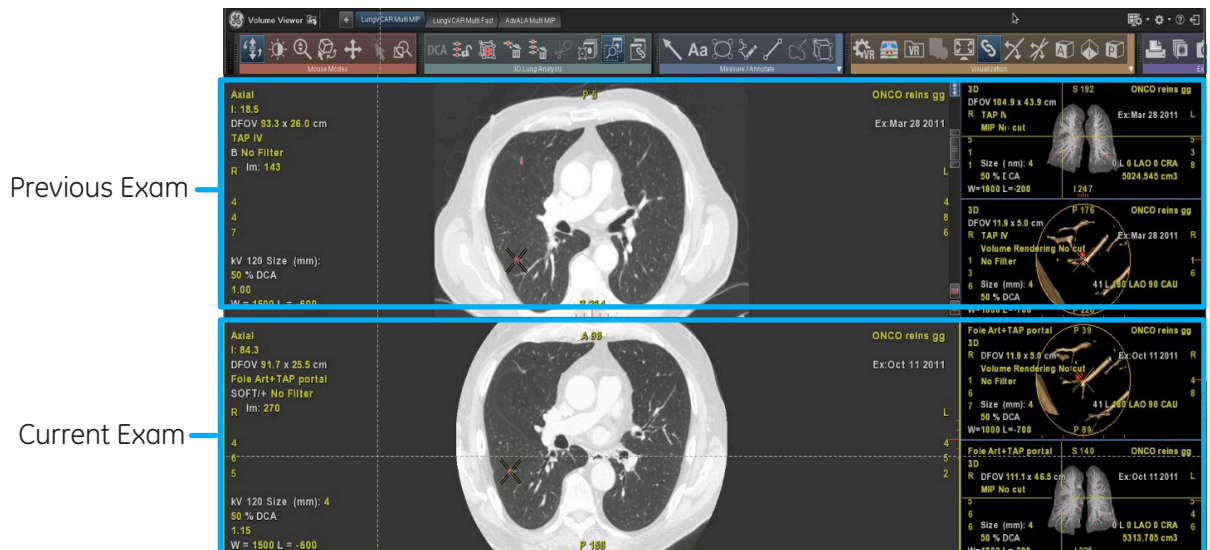


- In **Review** mode:

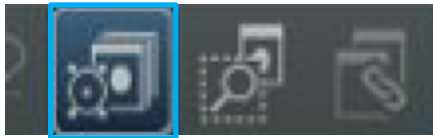


1. Deposit a bookmark by placing mouse on previous exam's lesions and pressing the **Insert** key (or by using the **Deposit bookmark** icon), AND/OR verify existing bookmarks.




2. If needed, adjust bookmark by placing mouse cursor on current exam's lesion and pressing the **Shift** key, then click **Confirm**, if necessary.

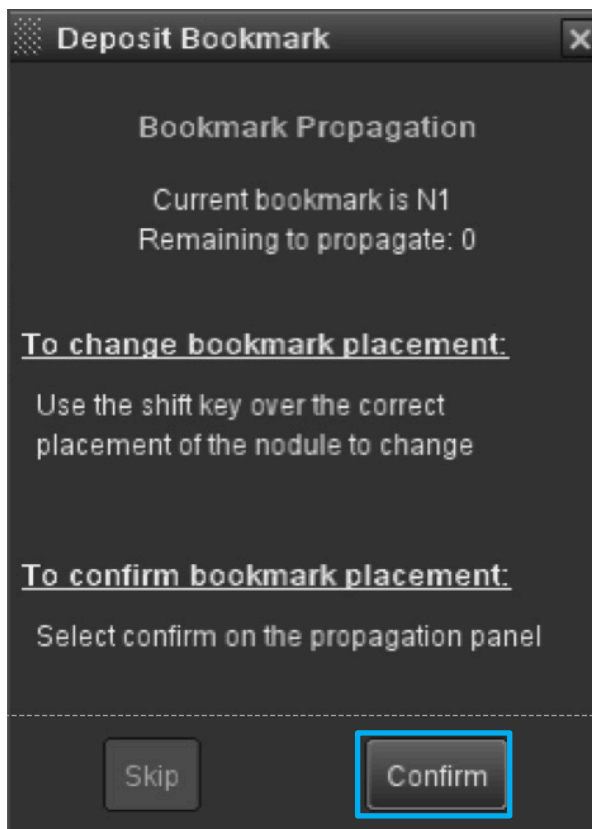

- Go to **Analysis** mode.

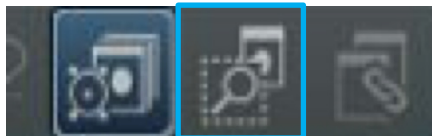

- Nodule(s) will be extracted with indication of Nodule Type: Solid (S), Partially Solid (PS), Non Solid (NS).
- Total volume and 2D distance measurements for A-P, L-R, and I-S appear on viewports.
- Statistics and growth will be displayed in the inferior of right corner of current exam's nodule's VR extraction.

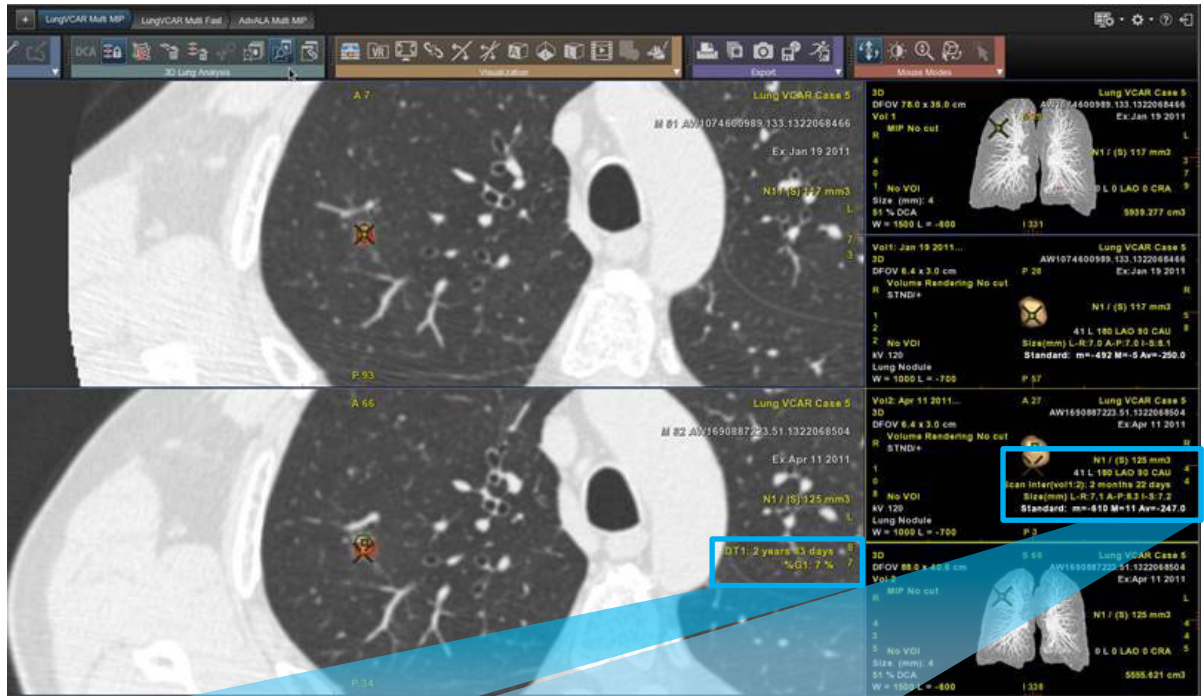


DT1: 2 years 83 days
%G1: 7 %

DT1: Doubling Time of Nodule 1

%G1: Percentage of growth for Nodule 1

N1 / (S) 125 mm³
41 L 180 LAO 90 CAU
Scan Inter(vol1:2): 2 months 22 days
Size(mm) L-R:7.1 A-P:8.3 I-S:7.2
Standard: m=-610 M=11 Av=-247.0

N1: Nodule 1 (Solid), Volume: 125 mm³
Scan Inter (vol 1:2): Time between the two compared exams
Standard: Nodule HU statistics.

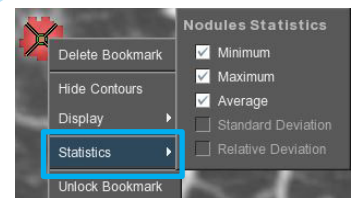

Statistics to be displayed can be selected by right clicking on **green bookmark** and selecting **statistics**.

- Upon exiting exam, a pop-up will appear with exit options to Save Report and **AutoSave State**.

WARNING! To assure an efficient and safe use of Volume Viewer Apps, it is essential for you to read the User Guides and the Customer Release Notes before attempting to start. The entire documentation is available on your system. Make certain that your documentation is readily available at all times. You shall under no circumstances use Volume Viewer Apps if the documentation is not available. If you need help, please contact the Online Center.

GE, and the GE Monogram are trademarks of General Electric Company.